


# Provent<sup>®</sup> Sleep Apnea Therapy:

## A New Option for treating Obstructive Sleep Apnea

### Prescribe Provent Therapy – It's treatment that sticks

Provent Sleep Apnea Therapy is a prescription-only therapy for the treatment of obstructive sleep apnea (OSA). It is an effective option for patients who are noncompliant with their prescribed CPAP.


- FDA cleared for obstructive sleep apnea
- Simple, discreet single-use device applied directly to the nostrils
- Well tolerated<sup>1-4</sup>
- High patient acceptance<sup>1-4</sup>
- Creates expiratory positive airway pressure (EPAP) to keep airway open
- Significantly decreases OSA-related snoring<sup>4</sup>

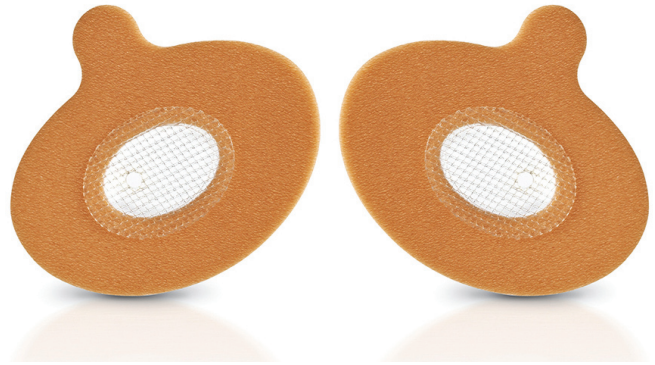

**The nasal EPAP device significantly reduced the AHI and improved subjective daytime sleepiness compared to the sham treatment in patients with mild to severe OSA with excellent adherence.**

Conclusion from Randomized trial of novel EPAP device for Treatment of OSA. Berry, Kryger, et al

### Provent Therapy works across all OSA severities

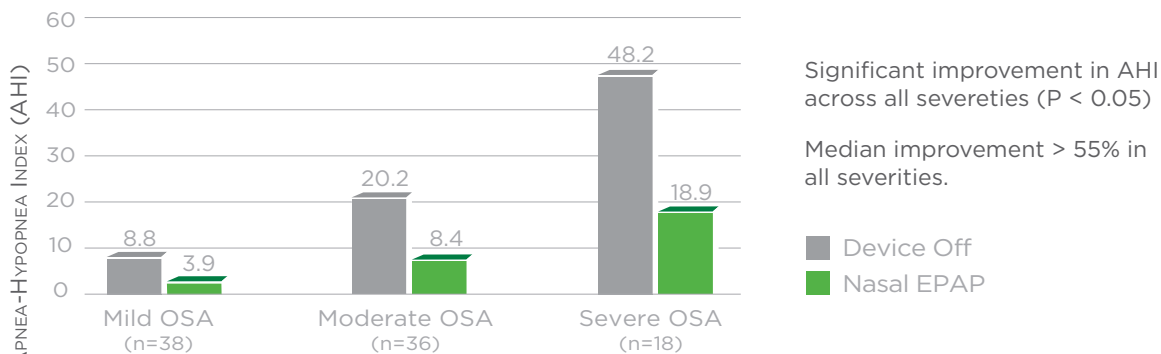

MEDIAN AHI FROM WEEK 1 PSG BY OSA SEVERITY.

Significant improvement in AHI across all severities (P < 0.05)

Median improvement > 55% in all severities.

■ Device Off  
■ Nasal EPAP


### Getting Acclimated

Like any new therapy, Provent Sleep Apnea Therapy takes some getting used to. Most patients will acclimate within a week and once acclimated, Provent Therapy shows high patient acceptance and compliance.

## How it works:

Provent's proprietary MicroValve technology uses the patient's own breathing to create Expiratory Positive Airway Pressure to increase airway caliber during the critical end-expiratory period when the airway is most narrow.

### Provent® Therapy Creates Expiratory Positive Airway Pressure (EPAP)

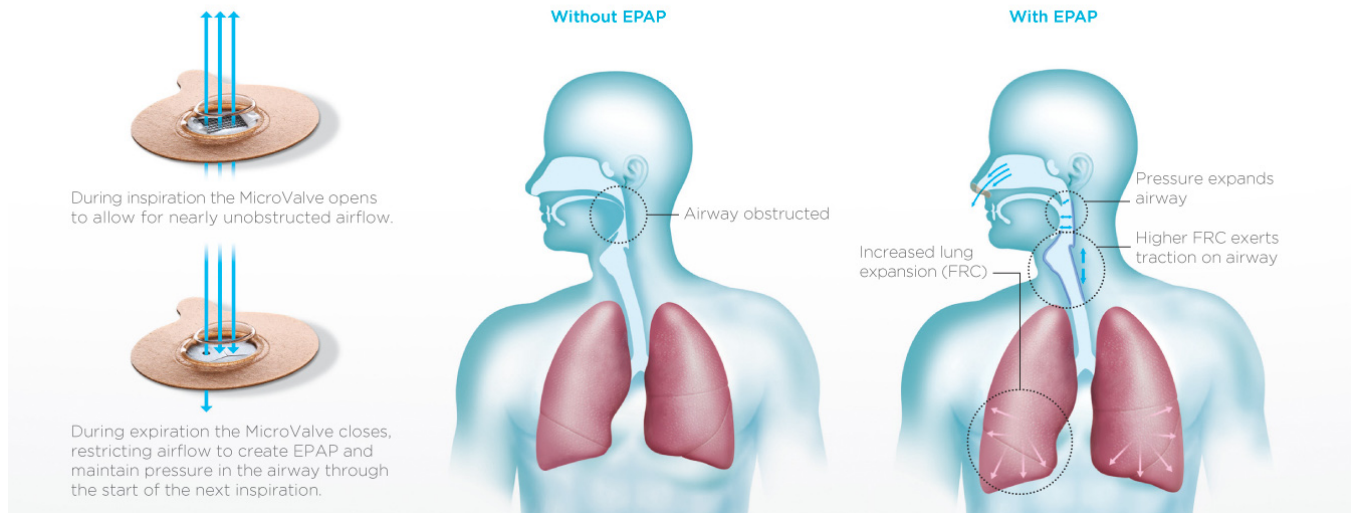

## How to Prescribe:

Provent Therapy is available by prescription through a national mail-order distributor and over 100 DME suppliers nationwide. Complete and FAX your prescription to your preferred supplier.

### 1. Complete prescription, including the following information:

- Provent Therapy
- Prescription duration (3 months, 12 months, etc.)

### 2. Fax prescription to Provent Therapy supplier of your choice:

(for a list of Provent Therapy suppliers, go to [www.proventtherapy.com/supplier\\_locator](http://www.proventtherapy.com/supplier_locator))

To contact a Ventus Medical representative, call **1-888-SLP-WELL Option 1**

For more information, visit: [www.proventtherapy.com](http://www.proventtherapy.com)

## References

1. Berry RB, Kryger MH, Massie CA. A novel nasal expiratory positive airway pressure (EPAP) device for the treatment of obstructive sleep apnea: a randomized controlled trial. *Sleep*. 2011;34:479-485.
2. Walsh JK, Griffin KS, Forst EH, et al. A convenient expiratory positive airway pressure nasal device for the treatment of sleep apnea in patients non-adherent with continuous positive airway pressure. *Sleep Med*. 2011;12:147-152.
3. Rosenthal L, Massie CA, Dolan DC, Loomas B, Kram J, Hart RW. A multicenter, prospective study of a novel nasal EPAP device in the treatment of obstructive sleep apnea: efficacy and 30-day adherence. *J Clin Sleep Med*. 2009;5:532-537.
4. Kryger MH, Berry RB, Massie CA. Long-term use of a nasal expiratory positive airway pressure (EPAP) device as a treatment for obstructive sleep apnea (OSA). *J Clin Sleep Med*. 2011;7:449-453.